



# Flowflex™

## SARS-CoV-2

### Antigen Rapid Test (Self-Testing)

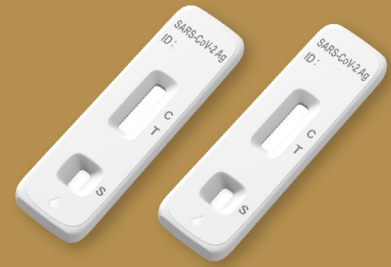

A rapid test for the detection of SARS-CoV-2 nucleocapsid antigens in anterior nasal swab specimens.  
For in vitro diagnostic use only. For self-testing.

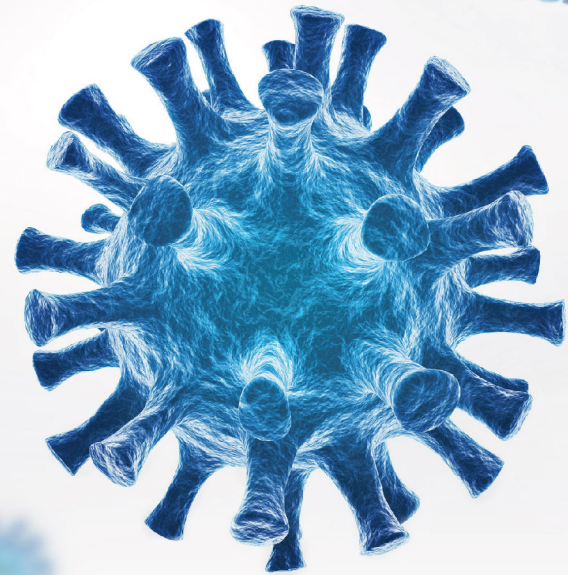

The Global Leader of Rapid Test with **26** Years Experience.



Fast



Accurate



Easy to Use



Reliable



CE Marked

**ACON®**

# Flowflex SARS-CoV-2 Antigen Rapid Test (Self-Testing)

The SARS-CoV-2 Antigen Rapid Test is a lateral flow test for the qualitative detection of the nucleocapsid antigen from SARS-CoV-2 in anterior nasal swab specimens directly from individuals suspected of COVID-19 within the first seven days of the onset of symptoms. The test can also test specimens from individuals without symptoms.

- **Specimen:** Anterior nasal swab specimens
- **Test Time:** Results at 15 min.
- **Shelf life:** 24 months
- **Storage temperature:** 2-30°C
- **Accuracy:** 98.8%
- **Sensitivity:** 97.1%
- **Specificity:** 99.5%



## Test Procedure


### 1. Test Preparation

#### A. Open your test kit:




#### B. You should have:

Note: For 1T and 5T, the hole is on the kit box.

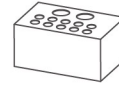


Tube Holder (for 25T only)

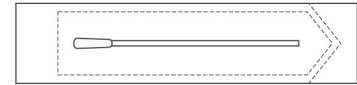

Disposable Swab



Test cassette

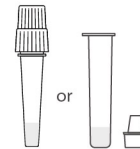

Extraction Buffer Tube



Package Insert




Waste Bag

#### C. Wash or sanitize your hands.



#### D. Read the instructions.

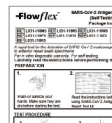


#### E. Check the expiration date.



#### F. Open the pouch.




#### G. Check the cassette

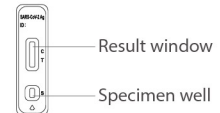


### 2. Specimen Collection


#### COLLECTION BY AN ADULT CAREGIVER

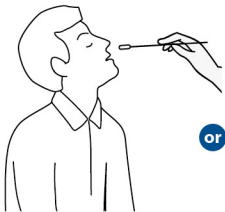


#### SELF COLLECTION

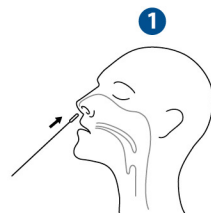


1 Insert the entire absorbent tip of the swab into one nostril. Using gentle rotation, push the swab less than 2.5 cm from the edge of the nostril.

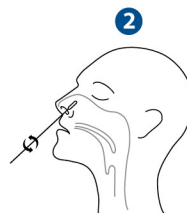

2 Rotate the swab 5 times brushing against the inside of the nostril.

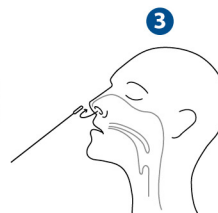


3 Remove the swab and insert it into the other nostril.

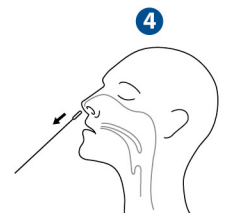

4 Remove swab from the nostril.

### 3. Sample Preparation and running the test

1

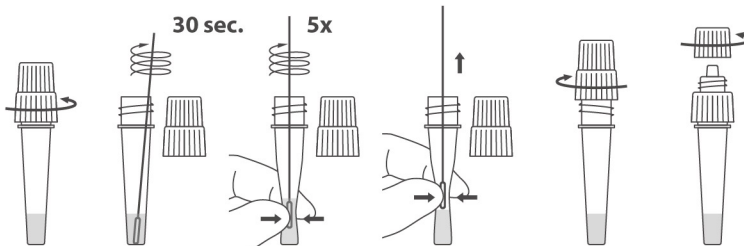


4 drops of the processed specimen




Sample Well

2

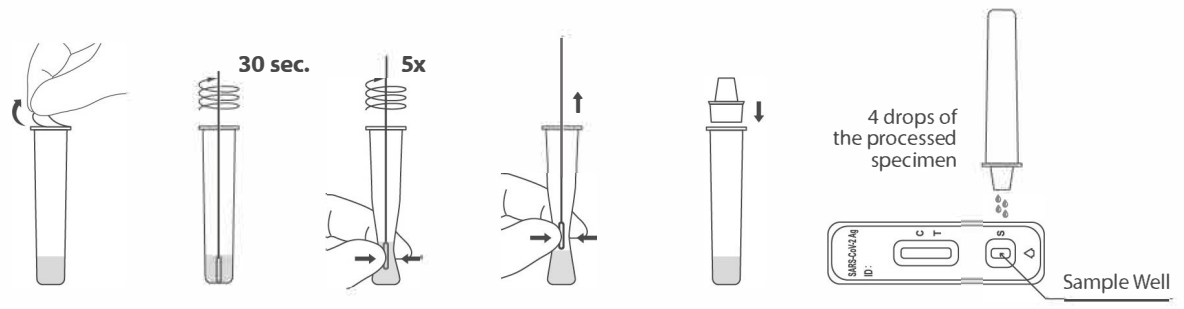

15-30 min.

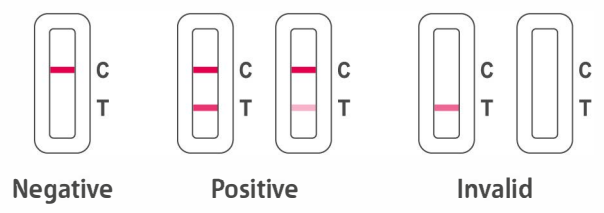

### Ordering Information

Product Name	Catalog No.	Format	Specimen	Package	PZN
Flowflex SARS-CoV-2 Antigen Rapid Test (Self-Testing)	L031-118M5 ✓	Cassette	Anterior nasal swabs	1 Test/Kit	17522524
	L031-118A5 ✓	Cassette	Anterior nasal swabs	5 Tests/Kit	17532379
	L031-118Q5 ✓	Cassette	Anterior nasal swabs	25 Tests/Kit	17522027

✓ CE Marked

