

**PRODUCT DESCRIPTIONS:** Blue Dot Digital Thermometer/Superdrug Digital Thermometer (Flexi-Tip)**PRODUCT CODES:** 30THERM3 252053**DEVICE DESCRIPTION:**

The Digital Thermometer is intended for the measurement of human body temperature, and is for home and hospital use.

**Intended Uses**

Measuring the body temperature of humans.

**Classification**

Class IIa non- sterile, according to Rule 10 of Annex IX, MDD/93/42 EEC

**Physical Performance and Dimension Specifications**

Digital Thermometer is intended for measuring human body temperature for clinical or home use on people of all ages, the measurement site is oral, armpit and rectal.

Component name	Specification
LCD size	15.5x6.5mm
Unit External Dimension	127x18x10mm
Display Range	32.0°C~42.0°C(89.6 °F -107.6 °F)
Accuracy	±0.1°C(35.5 °C -42.0 °C)/ ±0.2 °F (95.9 °F -107.6 °F) ±0.2°C(<35.5°C )/ ±0.4 °F (<95.9 °F)
Response	60Seconds
Battery	1.5V (LR/ SR41)

**Other Capabilities**

- a) Beeps when reaches high temperature;
- b) Convenient memory feature (Automatic recall of last temperature taken);
- c) Reads temperature in Fahrenheit;
- d) Automatic power off

**Storage/Shelf Life**

Stored appropriately, the product has a shelf life of 3 years in intact package.

**Cleaning and Disinfection**

The best way to clean the thermometer tip is by applying a disinfectant( eg 70% medical alcohol) with a damp cloth. It should be disinfected before each use. The thermometer can be immersed in liquid or luke warm water for thorough cleaning.